

SAVE II+™



Our goal is to give customers the most accurate information on all of our products. Due to significant global demand for the SAVE II+, please contact our Customer Support Department at the numbers or email below for the latest availability and pricing information.

855-428-6074 (option #2) TOLL FREE
704-705-1222 (option #2) PHONE
info@combatmedical.com

SKU: 25-220

PRODUCT DESCRIPTION

In response to COVID-19, the SAVE II underwent performance modifications to maximize efficiency. The SAVE II+ is authorized for emergency use in healthcare settings, subject to the conditions set forth in a US Food and Drug Administration Emergency Use Authorization (EUA) 200336.

The SAVE II+ is intended to provide ventilatory support for adults during CPR or when positive-pressure ventilation (PPV) is required to manage acute respiratory failure (ARF), or other situations where mechanical ventilation is needed. The SAVE II+ series is appropriate for adults that weigh at least 45 kg (99lbs). It is intended to be used in pre-hospital settings, field hospitals, outpatient environments, hospitals, ICU's, transport environments or any other healthcare setting requiring the use of a ventilator.

The SAVE II+ uses a battery-powered compressor to deliver air to a patient for up to 8.5 hours on a single charge. To support use in austere or supply constrained environments, where compressed oxygen is unavailable or ill advised, the device does not require compressed oxygen. However, FiO₂ can be increased from 21% to 100% if compressed oxygen or an oxygen concentrator is available.

SAVE II+ ENHANCED FEATURES:

- Increased maximum minute volume from 8LPM to 12LPM
- PEEP increased to 20 cmH₂O. (Up from a previous maximum of 10 cmH₂O)
- HME Filter and Exhale Bacterial Filter compatible
- Simplified Quick Start Guide

FEATURES OF ALL SAVe II SERIES PRODUCTS:

- Height-based adult presets enable rapid setup
- Presets dial in lung-protective ARDSnet-based parameters (~6 ml/kg) based on ideal body weight
- Small size and weight make it one of the most portable ventilators on the market
- Adjustable tidal volume (TV), respiratory rate (RR), peak inspiratory pressure (PIP) and positive end expiratory pressure (PEEP)
- Alarm dashboard and integrated quick reference chart simplifies troubleshooting
- No compressed gas required
- Delivers FiO₂ up to 100% using included oxygen reservoir and user provided flow regulated oxygen source.
- Displays breath-to-breath PEEP and PIP measurements

In accordance with Emergency Use Authorization 200336 from the FDA:

- i. The SAVell+ ventilator has not been FDA cleared or approved;
- ii. The SAVell+ ventilator has been authorized by FDA under an EUA;
- iii. The SAVell+ ventilator is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of ventilators under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner

TRAINING

[SAVe II+ Operator Manual](#)

[SAVe II+ Quick Start Guide](#)

SPECIFICATIONS

- PN: 25-220
- Currently not approved for use on Military Aircraft
- Set Weight: 9.9 lbs
- Set Dimensions: 6.5 x 6.25 x 2.0
- Shelf Life: 8 Year (SAVe II+ Unit) / 1 Year (Circuit)
- Made in USA
- Set Components:
 - 1 Hard Case
 - 1 SAVe II+ Unit
 - 1 Attenuator
 - 1 Charger
 - 1 Power Cord
 - 1 Users Manual
 - 1 Quick Start Guide

- 1 Circuit
- 1 HME Filter
- 1 Oxygen Reservoir