

CIRCULATION




BATTLE WRAP®


1 ea. - 6' x 4", clear, self-adhering, compression wrap



Step 1: ID and expose wound.

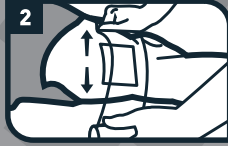

Step 2: Pack and/or cover wound with gauze. Center wrap over gauze.

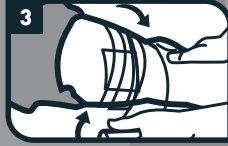

Step 3: Wrap firmly. Reassess.

REF 02-001 CMS-PUB-BW-04-006

NSN: 6510-01-612-8229

© 2017 Combat Medical Systems, LLC

Pat. Info: combatmedicalsistemas.com/ip

Prod. Info: combatmedicalsistemas.com/e-labels

TEAR

TEAR



Combat Medical

5555 Harrisburg Industrial Park Drive
Harrisburg, NC 28075 USA

1-855-428-6074

www.combatmedicalsistemas.com



entrotech
life sciences



Made in USA

WARNINGS: For use by trained medical providers. Once applied, patient should be monitored. Dispose of soiled material per biohazard waste guidelines.

