




EMERGENCY BLANKET

1 ea - 84" x 52" Mylar insulation blanket

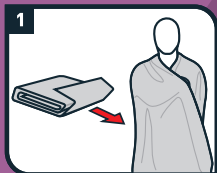

Step 1: Unroll blanket to full length, drape over body, hold close as possible to concentrate body heat.



Step 2: When possible, cover head and draw legs toward the chest to utilize and preserve body core temperature.



Step 3: For post-trauma shock, wrap blanket around victim and keep immobile. Get immediate medical help.



MARCH™
Medical Gear

WARNING: Keep away from open flame. Product will burn if ignited.

Made in USA



Pat. Info: combatmedicalsystems.com/ip
© 2015 Combat Medical Systems, LLC

REF 04-101 CMS-PUB-EB-04-003

www.combatmedicalsystems.com

1-855-428-6074

Harrisburg, NC 28075 USA

5555 Harrisburg Industrial Park Drive

Combat Medical
Distributed by:



HEAD & HYPOTHERMIA™