

# AIRWAY



# NASAL TRUMPET


1ea - 28Fr Robertazzi Nasopharyngeal Airway



**Step 1:** 👁 Measure fit tip-to-tip.



**Step 2:** Lubricate and insert. If resistance occurs, attempt in other nostril.

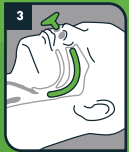

**Step 3:** Reassess. ↻



Distributed by:  
Combat Medical  
5555 Harrisburg Industrial Park Drive  
Harrisburg, NC 28075 USA  
1-855-428-6074  
[www.combatmedicalsyste.ms.com](http://www.combatmedicalsyste.ms.com)

REF 11-104 CMS-PUB-NTNL-04-001  
Pat. Info: [combatmedicalsyste.ms.com/ip](http://combatmedicalsyste.ms.com/ip)  
© 2016 Combat Medical Systems, LLC  
Made in Malaysia



STERILE R

**WARNINGS:** Federal (USA) law restricts this device to sale by or on order of a physician. Replace per CDC guidelines and/or hospital policy. Not made with natural rubber latex. For single use only. DO NOT resterilize.

LOT

