

◀ TEAR


TEAR ▶



# CELOX<sup>TM</sup> RAPID

1ea-sterile 3" x 5' Hemostatic Z-fold Gauze



**Step 1:**  ID and expose wound. Open package.



**Step 2:** Tightly pack the unfolding CELOX<sup>TM</sup> RAPID directly to the source of bleeding. Pack remaining wound cavity with CELOX<sup>TM</sup> RAPID.



**Step 3:** Apply FIRM pressure directly to the wound for at least 1 minute or until bleeding stops.



**Step 4:** Wrap firmly with pressure bandage. Reassess.

\* Additional instructions on back of package.

◀ TEAR

TEAR ▶



**MARCH<sup>TM</sup>**  
Medical Gear

## MASSIVE HEMORRHAGE

◀ TEAR



TEAR ▶

**Step 5:** Discard any remaining CELOX™ RAPID.

**Step 6:** Transfer patient to medical facility as soon as possible.

**Step 7:** Show empty pack to medical personnel.

### ATTENTION MEDICAL FACILITY PERSONNEL.

1. Physically remove CELOX™ RAPID from the wound and any loose surface granules.
2. Fully flood entire wound area with sterile saline irrigation solution.
3. Proceed with normal irrigation and / or suction.
4. Ensure all product is removed from the wound prior to initiation of wound treatment.



Distributed by:  
Combat Medical  
5555 Harrisburg Industrial Park Drive  
Harrisburg, NC 28075 USA  
1-855-428-6074  
[www.combatmedicalsistemas.com](http://www.combatmedicalsistemas.com)

[REF] 30-215 CMS-PUB-CLXR-04-001

Pat. Info: [combatmedicalsistemas.com/ip](http://combatmedicalsistemas.com/ip)  
© 2015 Combat Medical Systems, LLC

 **STERILE** **R** **R<sub>x</sub> ONLY**

## CELOX™

 Medtrade Products Ltd,  
Electra House, Crewe Business Park,  
Crewe, CW1 6GL, UK.  
+44 (0)1270 500019  
Artwork Ref: MTP-16-863  
[www.celoxmedical.com](http://www.celoxmedical.com)

**WARNINGS:** Device not intended for surgical use. Sterile-unless pouch is damaged or open. Do not eat. If ingested drink water to avoid any discomfort. Loss of sterility potentially poses a risk of infection. Do not apply over the eyes. If eye irritation occurs flush with water for 5 minutes. Contains chitosan.

◀ TEAR

TEAR ▶