

◀ TEAR

TEAR ▶



SENTINEL[®] CHEST SEAL

1ea – dual vent chest seal with Sentinel 10x™ adhesive; 1ea – wipe

◀ TEAR

TEAR ▶




MARCH[™]
Medical Gear

◀ TEAR


RESPIRATION



TEAR ▶

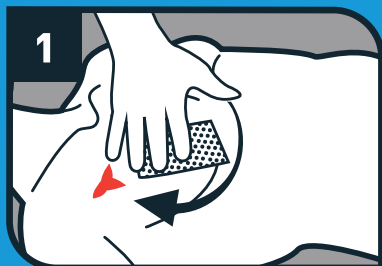

Step 1: ID and expose wound. Clean and dry wound area with enclosed wipe.

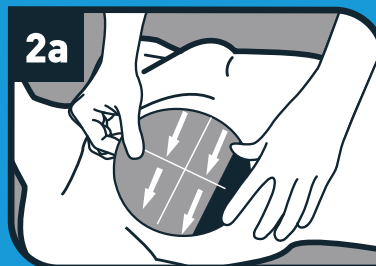

Step 2a: Remove Liner #1. Center the chest seal on the entrance wound. Secure adhesive #1 to the body.



Step 2b: Remove Liner #2 and apply firm pressure to ensure occlusive seal.



Step 3: Repeat steps 1 through 2b on the exit wound. Reassess. ↻



Combat Medical
5555 Harrisburg Industrial Park Drive
Harrisburg, NC 28075 USA
1-855-428-6074
www.combatmedicalsystems.com



REF 20-001 CMS-PUB-CS-04-008
NSN: 6510-01-600-4628

Pat. Info: combatmedicalsystems.com/ip
© 2015 Combat Medical Systems, LLC



Made in USA

WARNINGS: For use by trained medical providers. **DO NOT USE** if package is open or damaged. Once applied, patient should be monitored. Dispose of soiled material per biohazard waste guidelines.

◀ TEAR

TEAR ▶