



Combat Medical
5555 Harrisburg Industrial Park Drive
Harrisburg, NC 28075 USA
1-855-428-6074



www.combatmedicalsystems.com

REF 20-510 CMS-PUB-CTK-04-006
NSN: 6515-01-600-4971



Made in USA

Pat. Info: combatmedicalsystems.com/ip
© 2015 Combat Medical Systems, LLC

WARNINGS: For use by trained medical providers. Once applied, patient should be monitored. Dispose of soiled material per biohazard waste guidelines.



MARCH™
Medical Gear



SENTINEL® CHEST TRAUMA KIT

2ea-Sentinel Dual vent chest seals with Sentinel 10x™ extreme strength adhesive; 1ea-Dart sterile, 14 gauge x 3.25", needle / catheter for chest decompression; 1ea-DartTarget™; 1ea - wipe



1: ID and expose wound. Clean and dry wound area with enclosed wipe.




2a: Remove Liner #1. Center the chest seal on the entrance wound. Secure adhesive #1 to the body.



2b: Remove Liner #2 and apply firm pressure to ensure occlusive seal.

2c: Repeat steps 1 through 2b on the exit wound.

Reassess. ↻

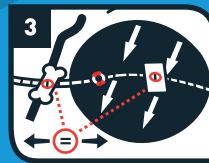


3a: Peel and place the DartTarget's bone shaped sticker mid-clavicle on the injured side.

3b: Males: Peel and place square tab on nipple.

Females: Peel and place square tab on the top center of breast.

3c: Slide Dart Target strip until numbers in windows match.

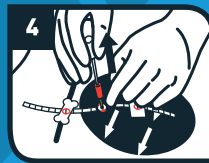

4: Use the MojoDart according to TCCC guidelines at the anatomical location indicated by the Dart Target. Insert the MojoDart at a 90° angle to the patient, ensuring placement above the rib.

Reassess. ↻

FOR PATIENTS WITH SIGNS AND SYMPTOMS OF A TENSION PNEUMOTHORAX

RESPIRATION



TWIST OPEN