

FOXSEAL™

The chest seal you can trust.

Crease-Free Seal

THE DEPENDABLE CHEST SEAL IN A COMPACT PACK

Foxseal™ Chest Seal is a pioneering occlusive dressing for the treatment of open chest wounds. Foxseal™ is designed for simple, consistent performance in the field as a tool in prevention and treatment of pneumothorax.

The advanced adhesive works on wet or hairy skin. Foxseal™ is a compact size to fit neatly in Individual First Aid Kits, so that leaks from creased dressings that are stored folded are no longer an issue. Two seals per pack allow entry and exit wounds to be treated and the design is valve-free. The chest can be vented with the easily-gripped tab and the adhesive re-seals. Foxseal™ provides dependable adhesion in the most challenging situations.

Foxseal™ provides dependable adhesion in the most challenging situations.

- Strong adhesion in extreme conditions
- Two per pack for ballistic wounds
- Compact, fits IFAK without creasing or folding
- Simple no-valve design
- Vent with large, easy to operate tab

Foxseal™ Specifications:


- Sterile
- Latex-free
- Two seals per pack
- FDA and CE listed



Adhesion at 55°C (131°F)



Adhesion at -18°C (0°F)

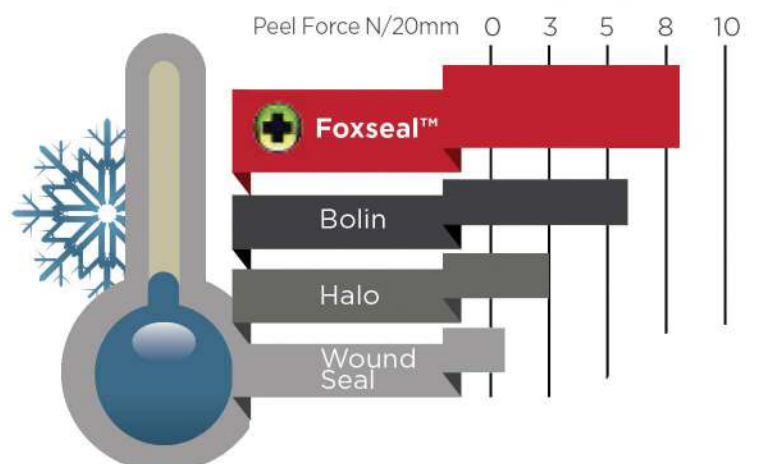

FOXSEAL™ The chest seal you can trust.



Instructions for use...

1. Wipe dirt/fluid from skin around wound.
2. Grip tab and peel to remove one Foxseal™ from clear liner. This exposes adhesive side.
3. Place dressing centered on the wound, adhesive side down. Preferably do this while patient exhales.
4. Firmly press dressing to skin around wound to seal.
5. If another wound is present, repeat steps 1-4 with 2nd seal.

Name

Item Code

National Stock Number

DIMENSIONS

Foxseal™

FG08814441



COMBAT MEDICAL
SIMPLIFY TACTICAL MEDICINE

5555 Harrisburg Industrial Park Drive
Harrisburg, NC 28075

combatmedicalsystems.com
855-428-6074 TOLL FREE