



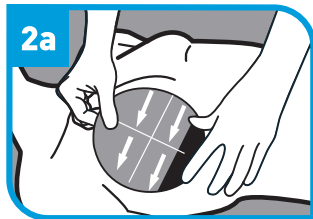


# SENTINEL® CHEST SEAL

## QUICK START CARD

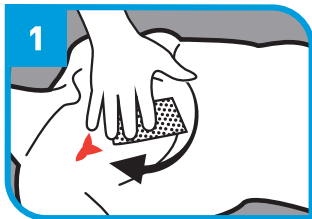

**Step 1:**  ID and expose wound.  
Clean and dry wound area with  
enclosed wipe.



**Step 2a:** Remove Liner #1. Center  
the chest seal on the entrance wound.  
Secure adhesive #1 to the body.



**Step 2b:** Remove Liner #2 and  
apply firm pressure to ensure  
occlusive seal.



**Step 3:** Repeat steps 1 through  
2b on the exit wound.  
Reassess. 

RESPIRATION





# SENTINEL<sup>®</sup> CHEST SEAL

## QUICK REFERENCE CARD

### PURPOSE:

This device is a rapid means to seal an open chest wound from penetrating chest trauma; such as a gun shot or stab wound.



- Wipe off blood and dirt from the wound.
- Apply as the patient exhales.
- Align arrows toward the ground.
- Look for exit wounds and treat.
- Place the patient injured side down.
- Reassess every 3-5 minutes.

This Quick Reference Card is provided as supplemental information in the event the original Instructions for Use is lost. Complete product information and disclosures are available at: [combatmedicalsyste.ms.com/e-labels](http://combatmedicalsyste.ms.com/e-labels)



REF 20-001-QR-EN

© 2016 Combat Medical Systems, LLC  
[www.combatmedicalsyste.ms.com](http://www.combatmedicalsyste.ms.com)