

TEAR ▶

Pat. Info: combatmedicalsyste.ms.com/ip © 2016 Combat Medical Systems, LLC

REF 31-105 CMS-PUB-SWAT-04-002

www.combatmedicalsyste.ms.com

1-855-428-6074

Harrisburg, NC 28075 USA

5555 Harrisburg Industrial Park Drive

Combat Medical

Distributed by:



TM



Made in USA



Product is not made with natural rubber latex.

www.swat-leu

Tostedt, Germany

Niedersachsenstrasse 70

OpBase GmbH

◀ TEAR

TEAR ▶

MASSIVE HEMORRHAGE



TM

◀ TEAR

TEAR ▶




# SWAT-T™

1ea-stretch-wrap-and-tuck tourniquet


◀ TEAR



**Step 1:** Remove from package and apply between wound and heart.

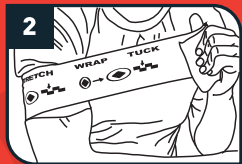

**Step 2:** Wrap tightly while pulling tension. Apply until the bleeding stops.

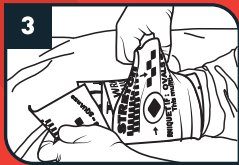

**Step 3:** Secure by tucking the end under itself. Reassess. ↻

◀ TEAR

TEAR ▶