


ONE-HANDED OPERATION INSTRUCTIONS

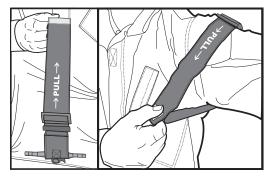


Hold the end of the strap by the white time label tab, allow the TMT to fall open with the word "PULL" facing away from you. Place around the injured extremity high and tight above the wound.

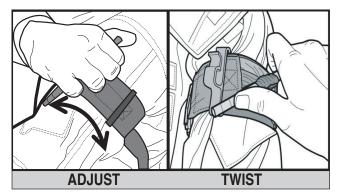


Adjust/rotate the TMT on extremity for ease of access to the torsion bar. Twist torsion bar until bleeding stops and the pulse is not present.

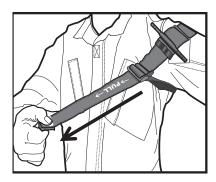


Pull time tab until Velcro® Brand material tail is separated from the TMT.

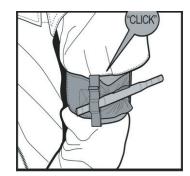

Lock the torsion bar into the retainer. Secure excess strap with Velcro® Brand material.

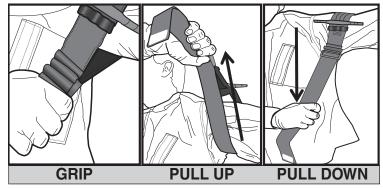

Grip tail close to the buckle, firmly pull up, out and away from the body and then downward in a quick ratcheting motion. Repeat step until snug around extremity.

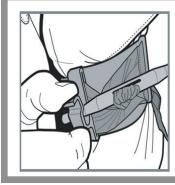

TO RELEASE/REMOVE THE TMT:

Unwrap excess tail from the Velcro® Brand material, put positive pressure on torsion bar. apply upward pressure on the retaining clip and push torsion bar out, then unhook buckle.

REMOVAL





5555 Harrisburg Industrial Park Drive

www.combatmedicalsystems.com

REF 31-115 | CMS-PUB-TMT-04-003

NSN: 6515-01-656-6191 Pat Info: combatmedical systems com/ir Alphapointe 3611 14th Avenue Brooklyn, NY 11218 USA






EC REP Savior Global 8 Ash Gardens Hamworthy, Poole, Dorset **BH15 4DE United Kingdom**

in one-handed


FDA Registered Patent Pending This is a SINGLE USE device. ISO 9001:2015 Certified

Limitation of liability: Alphapointe, its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application and use of the TMT. ONLY use the TMT as directed in the instructions for use. Do not remove the slider from the strap. The user assumes all risk liability. It is recommended to receive appropriate first aid training. Standard Use Guidelines: For life threatening hemorrhage situations. For the military in the Care Under Fire situation, treatment normally consists of using a tourniquet immediately to stop major bleeding of the extremities. In field care situation, the trained and authorized rescuer will perform all treatment necessary using the TMT as a last resort and should only be applied when bleeding cannot be stopped and the situation is life threatening.

TWO-HANDED OPERATION INSTRUCTIONS

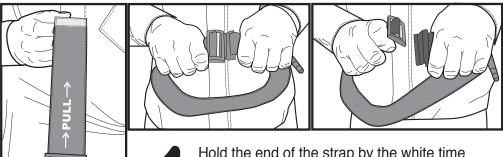


Hold the end of the strap by the white time label tab, allow the TMT to fall open. Grab TMT by the buckle and separate the two sections.

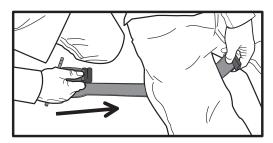


Wrap/Feed the TMT around the injured extremity high and tight above the wound, ensuring device is not twisted. Reconnect the buckle.



Pull time tab until VELCRO® Brand material is separated.

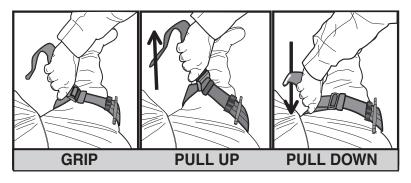

Grip tail close to the buckle, firmly pull up, out and away from the body and then downward in a quick ratcheting motion. Repeat step until snug around extremity.

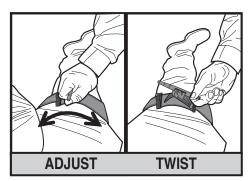

Adjust/rotate the TMT on extremity for ease of access to the torsion bar. Twist torsion bar until bleeding stops and the pulse is not present.

Torsion Bar



Lock the torsion bar into the retainer. Secure excess strap with VELCRO® Brand material.



Combat Medical
5555 Harrisburg Industrial Park Drive
Harrisburg, NC 28075 USA
1-855-428-6074

www.combatmedicalsystems.com
REF 31-115 | CMS-PUB-TMT-04-003

NSN: 6515-01-656-6191
Pat. Info: combatmedicalsystems.com/ip

Alphapointe 3611 14th Avenue Brooklyn, NY 11218 USA CAGE Code: 056F9



VELCRO® Brand Material

FDA Registered T
Patent Pending IS

Buckle

This is a SINGLE USE device ISO 9001:2015 Certified

Time Label Tab

Receiver

Retaining

9 Line MEDEVAC Request

Line 1: Location of the pick-up site

Line 2: Radio frequency, call sign, and suffix

Line 3: Number of patients by precedence

Line 4: Special equipment required

Line 5: Number of patients

Line 6: Security at pick-up site

Line 7: Method of marking pick-up site

Line 8: Patient nationality and status

Line 9: NBC Contamination (Peacetime: Terrain)